


















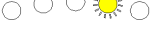




Parameter Table for Freedom Powerchair

1. With the power off press the Plus and Minus buttons at the same time then press Power.
2. The Power LED's show the level for each item. Use the plus & minus buttons to increase or decrease the levels.
3. Use the Horn button to move down to the next item.
4. When finished turn the power off to reset the program.

Num ber	SPEED LEDs	Value	Parameter Description
0		5	Forward speed.
1		5	Reversing speed.
2		5	Turning speed
3		5	Max. acceleration speed above 5km/h.
4		5	Max. acceleration speed between 2-5km/h.
5		5	Max. acceleration at a low speed below 2km/h.
6		5	Max. deceleration speed above 5km/h.
7		5	Max. deceleration speed between 2-5km/h..
8		5	Max. deceleration speed below 2km/h.
9		5	Turning speed when the wheelchair travels at a low speed.
10		5	Preset parameter of the wheelchair. The default value is 5.
11		5	Turning acceleration speed when the wheelchair turns.
12		5	Turn-back acceleration speed.
13		5	Alarm setting in case of solenoid valve failure 6-8 no alarm is made in case of solenoid valve failure
14		5	Turning of left motor -1-5 normal 6-8 reverse

15		5	Turning of right motor 1-5 normal 6-8 reverse
16		5	Fine tune direction & speed of left motor.
17		5	Fine tune direction & speed of right motor.
18		5	Control of solenoid valve delay time. The higher the value, the more it delays.
19		5	Adjustment of motor level. 1-5 4-level magnetic steel motor 6-8 8-level magnetic steel motor
20		2	Reversing warning volume, 1 is mute, 8 is max
21		5	Horn volume, 1 is mute, 8 is max
22		2	Key press beep volume, 1 is mute, 8 is max