

Seating System Armrests IPB, International

Illustrated Parts Breakdown

USA - Quantum Rehab

401 York Avenue
Duryea, Pa 18642
www.quantumrehab.com

USA - Pride Mobility Products Corp.

182 Susquehanna Avenue
Exeter, Pa 18643
www.pridemobility.com

Canada

5096 South Service Road
Beamsville, Ontario L0R 1B3
www.pridemobility.ca
www.quantumrehab.com

Australia

20-24 Apollo Drive
Hallam, Victoria 3803
www.pridemobility.com.au
www.quantumrehab.com.au

New Zealand

38 Lansford Crescent
Avondale, Auckland 0600
www.pridemobility.co.nz

UK

32 Wedgwood Road
Bicester, Oxfordshire OX26 4UL
www.pride-mobility.co.uk
www.quantumrehab.co.uk

Netherlands

(Authorised EU Representative)

De Zwaan 3
1601 MS Enkhuizen
www.pridemobility.eu
www.quantumrehab.eu

Italy

Via del Progresso, ang. Via del Lavoro
Loc. Prato della Corte
00065 Fiano Romano (RM)
www.pride-italia.it
www.quantumrehab-italia.it

France

26 Rue Monseigneur Ancel
69800 Saint-Priest
www.pridemobility.fr
www.quantumrehab.fr

Spain

Calle Las Minas Número 67
Polígono Industrial Urtinsa II, de Alcorcón
28923 Madrid

Germany

Hövelrieger Str. 28
33161 Hövelhof

China

Room 508, Building #4
TianNa Business Zone
No. 500 Jianyun Road
Pudong New Area
Shanghai 201318




Back Cane Mounted

Desk Length, Left / Right, Back Cane Mounted Cantilever Armrest	1
Full Length, Left / Right, Back Cane Mounted Cantilever Armrest	3
Appendix A:Pages Index	5
Appendix B:Parts Index	6

Back Cane Mounted

*** Individual components are not available for replacement. The entire assembly must be purchased. ***

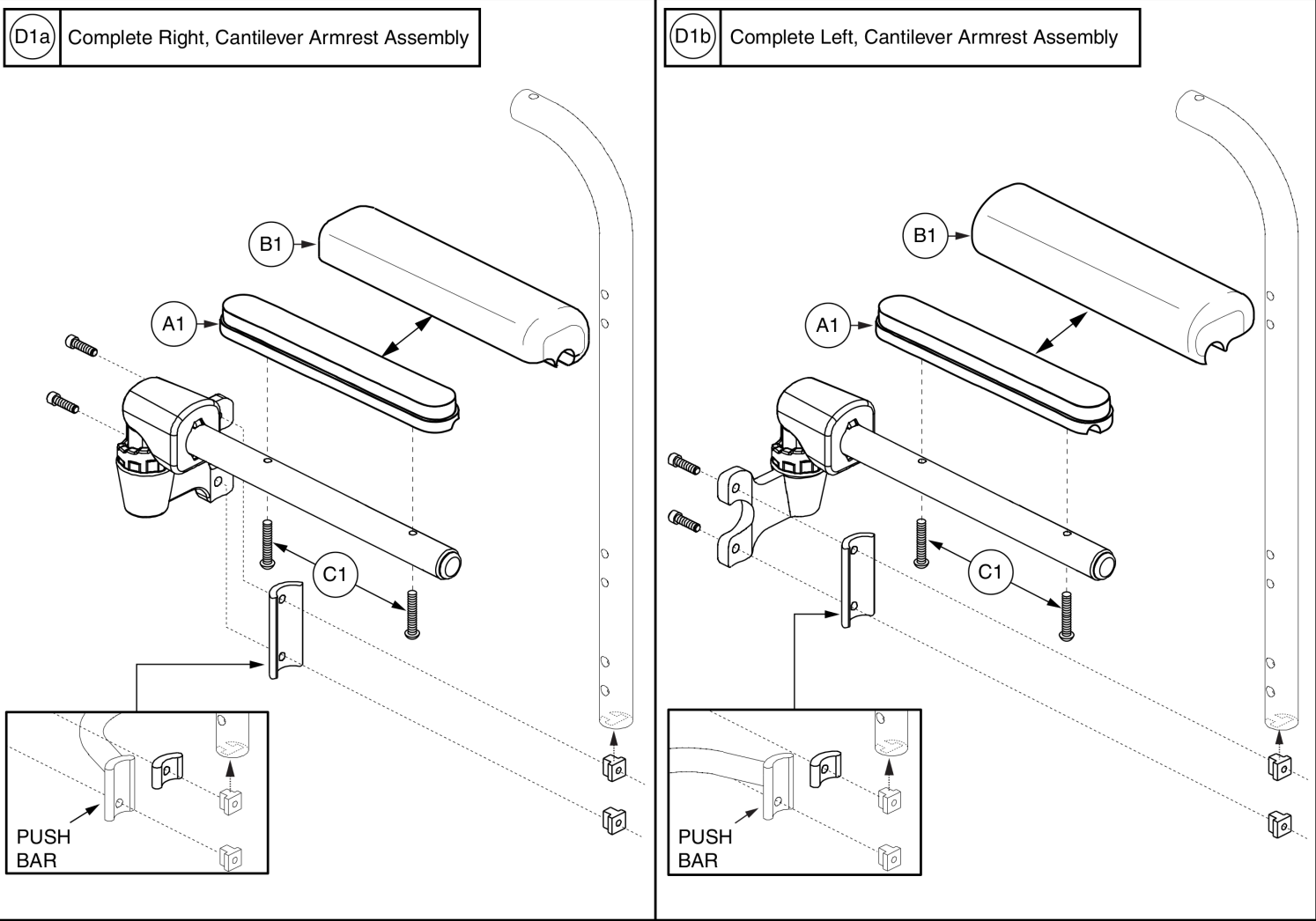




Item	Part No.	Name	Qty
A1	SETLRAM1115	ARMREST,PAD,SHORT,QUANTUM,V2	1
B1	SETLRAM1164	ARMPAD,ASSY,WATERFALL,BLACK,9", DESK LENGTH	1
C1	SCRASBH1053	SCREW,BHSCS,BLACK ZINC,10-32 X 1.25"	2
D1a	FRMASMB16191	ARMREST,CANTILEVER ARMS,DESK LENGTH,RIGHT	1
D1b	FRMASMB16190	ARMREST,CANTILEVER ARMS,DESK LENGTH,LEFT	1

Back Cane Mounted

*** Individual components are not available for replacement. The entire assembly must be purchased. ***

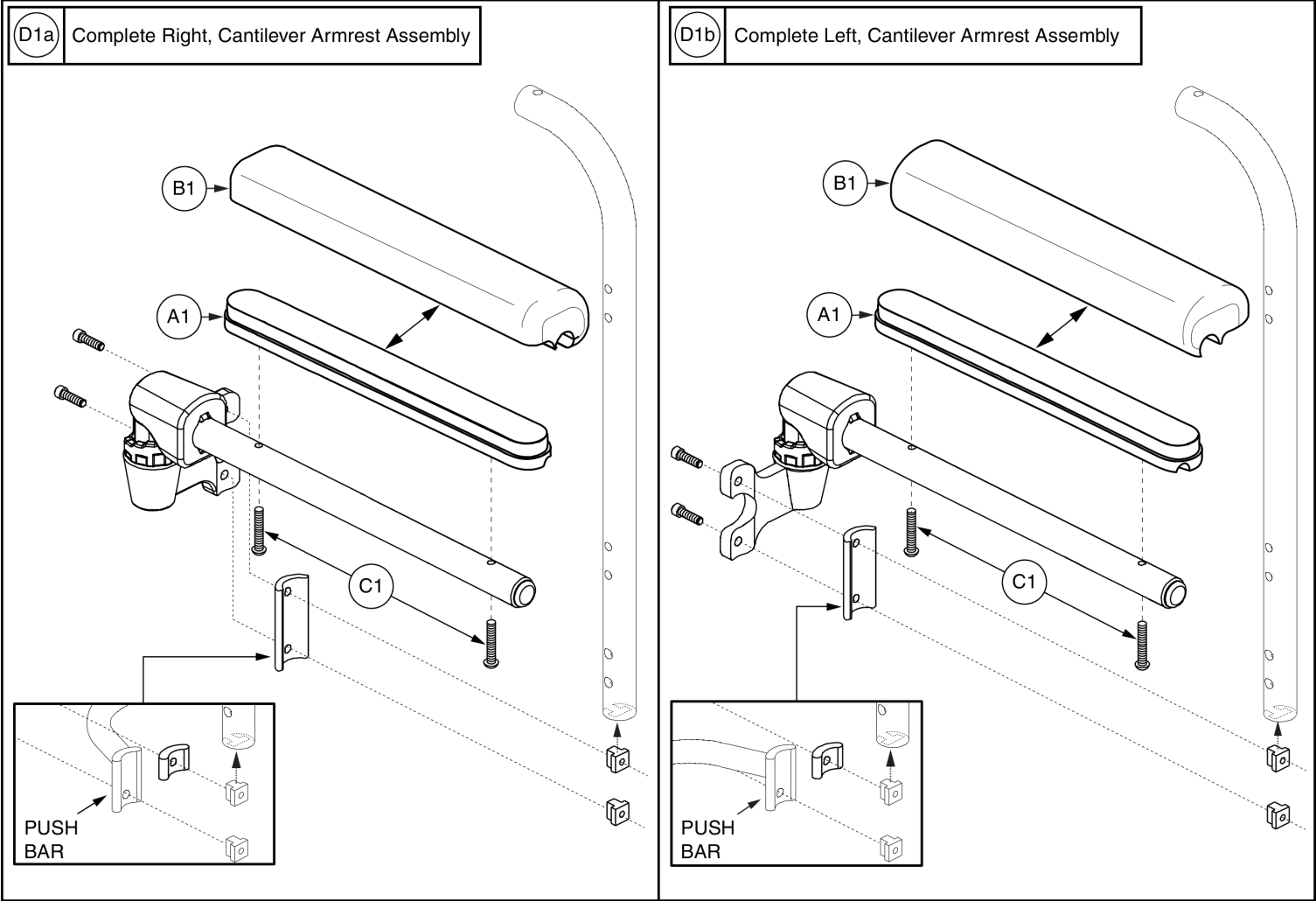



Item	Part No.	Name	Qty
A1	SETLRAM1116	ARMREST,PAD,LONG,QUANTUM,V2	1
B1	SETLRAM1165	ARMPAD,ASSY,WATERFALL,BLACK,14", FULL LENGTH	1
C1	SCRASBH1053	SCREW,BHSCS,BLACK ZINC,10-32 X 1.25"	2
D1a	FRMASMB16189	ARMREST,CANTILEVER ARMS,FULL LENGTH,RIGHT	1
D1b	FRMASMB16188	ARMREST,CANTILEVER ARMS,FULL LENGTH,LEFT	1

Appendix A:Pages Index

Desk Length, Left / Right, Back Cane Mounted Cantilever Armrest	1
Full Length, Left / Right, Back Cane Mounted Cantilever Armrest	3

Appendix B:Parts Index

FRMASMB16188	4
FRMASMB16189	4
FRMASMB16190	2
FRMASMB16191	2
SCRASBH1053	2, 4
SETLRAM1115	2
SETLRAM1116	4
SETLRAM1164	2
SETLRAM1165	4

Illustrated Parts Breakdown

USA - Quantum Rehab

401 York Avenue
Duryea, Pa 18642
www.quantumrehab.com

USA - Pride Mobility Products Corp.

182 Susquehanna Avenue
Exeter, Pa 18643
www.pridemobility.com

Canada

5096 South Service Road
Beamsville, Ontario L0R 1B3
www.pridemobility.ca
www.quantumrehab.com

Australia

20-24 Apollo Drive
Hallam, Victoria 3803
www.pridemobility.com.au
www.quantumrehab.com.au

New Zealand

38 Lansford Crescent
Avondale, Auckland 0600
www.pridemobility.co.nz

UK

32 Wedgwood Road
Bicester, Oxfordshire OX26 4UL
www.pride-mobility.co.uk
www.quantumrehab.co.uk

Netherlands

(Authorised EU Representative)

De Zwaan 3
1601 MS Enkhuizen
www.pridemobility.eu
www.quantumrehab.eu

Italy

Via del Progresso, ang. Via del Lavoro
Loc. Prato della Corte
00065 Fiano Romano (RM)
www.pride-italia.it
www.quantumrehab-italia.it

France

26 Rue Monseigneur Ancel
69800 Saint-Priest
www.pridemobility.fr
www.quantumrehab.fr

Spain

Calle Las Minas Número 67
Polígono Industrial Urtinsa II, de Alcorcón
28923 Madrid

Germany

Hövelrieger Str. 28
33161 Hövelhof
www.pridemobility.de

China

Room 508, Building #4
TianNa Business Zone
No. 500 Jianyun Road
Pudong New Area
Shanghai 201318

