

Seating System Armrests IPB, International

Illustrated Parts Breakdown

USA - Quantum Rehab

401 York Avenue
Duryea, Pa 18642
www.quantumrehab.com

USA - Pride Mobility Products Corp.

182 Susquehanna Avenue
Exeter, Pa 18643
www.pridemobility.com

Canada

5096 South Service Road
Beamsville, Ontario L0R 1B3
www.pridemobility.ca
www.quantumrehab.com

Australia

20-24 Apollo Drive
Hallam, Victoria 3803
www.pridemobility.com.au
www.quantumrehab.com.au

New Zealand

38 Lansford Crescent
Avondale, Auckland 0600
www.pridemobility.co.nz

UK

32 Wedgwood Road
Bicester, Oxfordshire OX26 4UL
www.pride-mobility.co.uk
www.quantumrehab.co.uk

Netherlands

(Authorised EU Representative)

De Zwaan 3
1601 MS Enkhuizen
www.pridemobility.eu
www.quantumrehab.eu

Italy

Via del Progresso, ang. Via del Lavoro
Loc. Prato della Corte
00065 Fiano Romano (RM)
www.pride-italia.it
www.quantumrehab-italia.it

France

26 Rue Monseigneur Ancel
69800 Saint-Priest
www.pridemobility.fr
www.quantumrehab.fr

Spain

Calle Las Minas Número 67
Polígono Industrial Urtinsa II, de Alcorcón
28923 Madrid

Germany

Hövelrieger Str. 28
33161 Hövelhof

China


Room 508, Building #4
TianNa Business Zone
No. 500 Jianyun Road
Pudong New Area
Shanghai 201318

2.1. Back Cane Mounted, Flip-up Arms

Desk Length, Left / Right, Flip Up Armrest, Textured	1
Full Length, Left / Right, Flip Up Armrest, Textured	3
Back Center Tube - 13"W-24"W, Textured	5
Appendix A:Pages Index	7
Appendix B:Parts Index	8

2.1. Back Cane Mounted, Flip-up Arms

*** Only items with callouts are available for replacement. ***

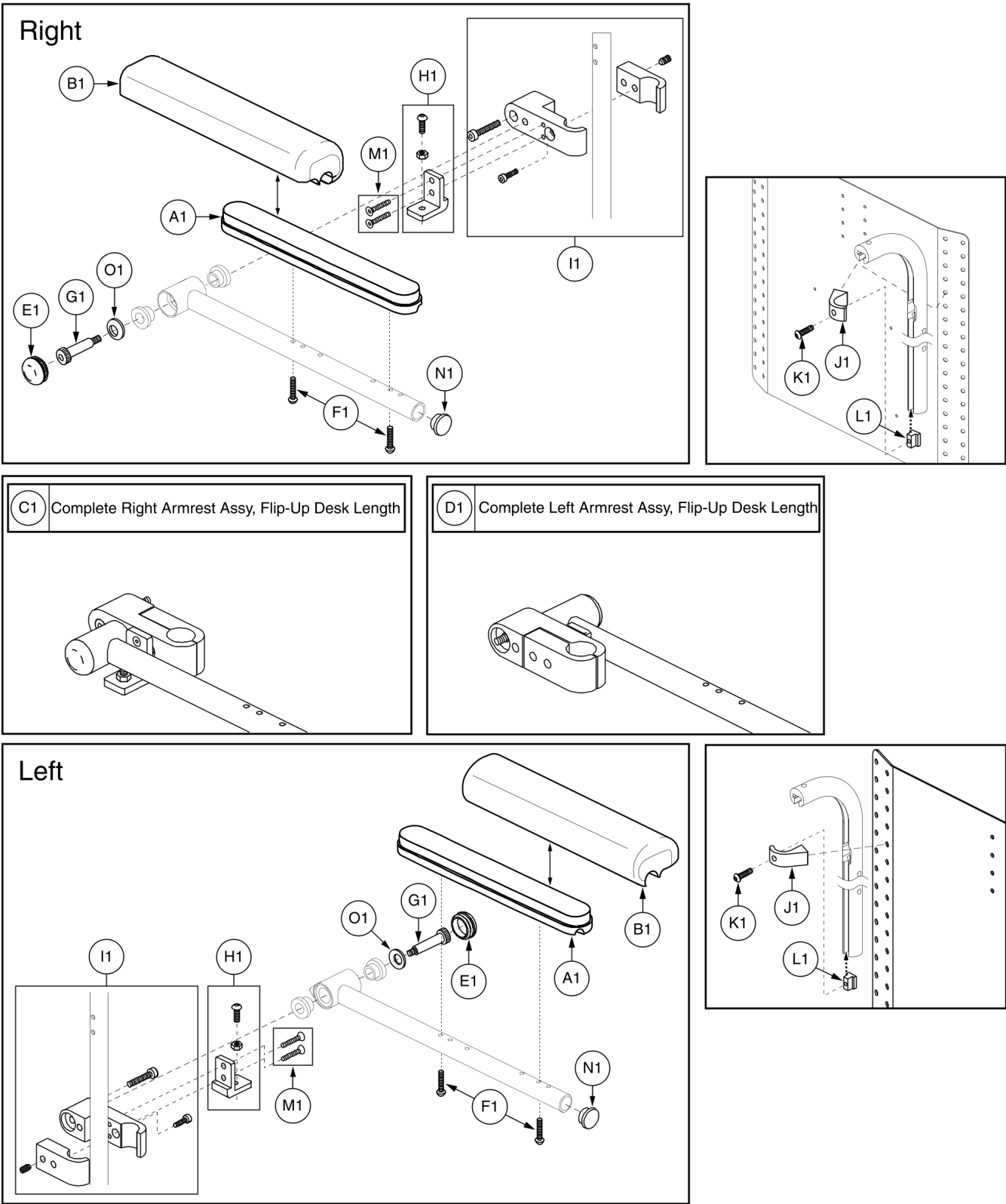


Both assemblies contain identical parts. There are left and right sides for assembly purposes.

Item	Part No.	Name	Qty
A1	SETLRAM1115	ARMREST,PAD,SHORT,QUANTUM,V2	1
B1	SETLRAM1164	ARMPAD,ASSY,WATERFALL,BLACK,9", DESK LENGTH	1
C1	FRMASMB7041	ARMREST,ASSY,FLIP-UP,DESK LENGTH,SYNERGY/VERSA TILT,RIGHT,TEXTURED	1
D1	FRMASMB7040	ARMREST,ASSY,FLIP-UP,DESK LENGTH,SYNERGY/VERSA TILT,LEFT,TEXTURED	1
E1	PROECAP1029	END CAP,PLASTIC,1.25" DIA,FITS 11 GAUGE TUBE,RIBBED,C-1005-058	1
F1	SCRASBH1053	SCREW,BHSCS,BLACK ZINC,10-32 X 1.25"	2
G1	BLTASHL1017	BOLT,SHOULDER,1/2" X 1.50"	1
H1	FRMASMB7042	ARMREST,ASSY,STOP BRACKET,FLIP-UP ARMREST,SYNERGY/VERSA TILT,TEXTURED	1
I1	FRMASMB7045	MOUNT BRACKET,ASSY,CLAMP,FLIP-UP ARMREST,SYNERGY/VERSA TILT,TEXTURED	1
J1		BACK PLATE BRACKET,T-SLOT,1",C-1359-005	2
K1	SCRASBH1148	SCREW,BHSCS,BLACK ZINC,1/4-20 X 0.50",W/PATCH	2
L1	DWR1359B003	T-NUT,BLACK,1/4-20 X 0.563",0.50" TH,B-1359-003	2
M1	SCRSHFH1010	SCREW,SHFH,BLACK ZINC,1/4-20 X 0.75"	2
N1	PROECAP1029	END CAP,PLASTIC,1.25" DIA,FITS 11 GAUGE TUBE,RIBBED,C-1005-058	1
O1	WASWAVY1007	WASHER,BELLEVILLE,0.515" ID X 1.00" OD X 0.035" TH	1

2.1. Back Cane Mounted, Flip-up Arms

*** Only items with callouts are available for replacement. ***

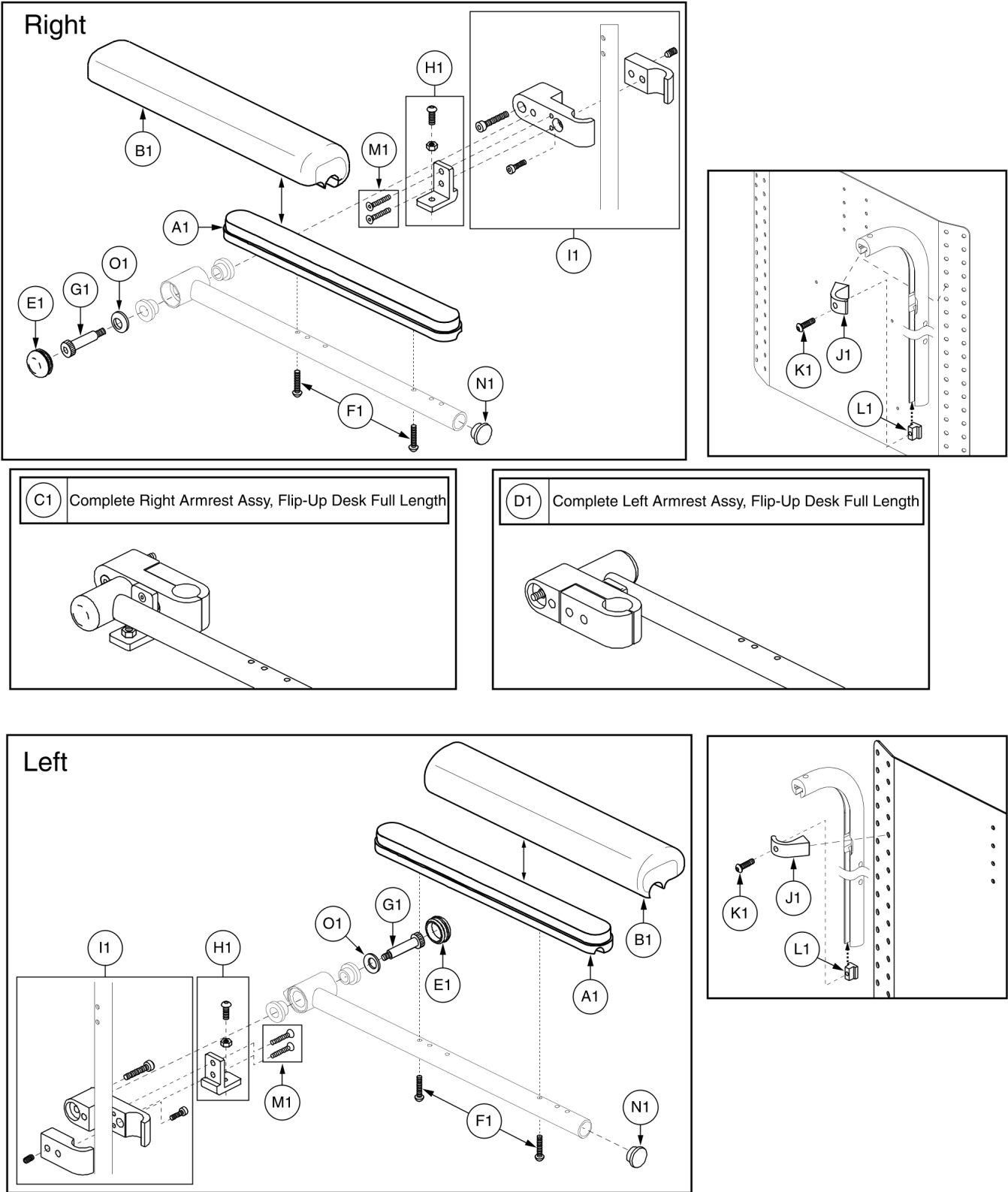


Both assemblies contain identical parts. There are left and right sides for assembly purposes.

Item	Part No.	Name	Qty
A1	SETLRAM1116	ARMREST,PAD,LONG,QUANTUM,V2	1
B1	SETLRAM1165	ARMPAD,ASSY,WATERFALL,BLACK,14", FULL LENGTH	1
C1	FRMASMB7039	ARMREST,ASSY,FLIP-UP,FULL LENGTH,SYNERGY/VERSA TILT,RIGHT,TEXTURED	1
D1	FRMASMB7038	ARMREST,ASSY,FLIP-UP,FULL LENGTH,SYNERGY/VERSA TILT,LEFT,TEXTURED	1
E1	PROECAP1029	END CAP,PLASTIC,1.25" DIA,FITS 11 GAUGE TUBE,RIBBED,C-1005-058	1
F1	SCRASBH1053	SCREW,BHSCS,BLACK ZINC,10-32 X 1.25"	2
G1	BLTASHL1017	BOLT,SHOULDER,1/2" X 1.50"	1
H1	FRMASMB7042	ARMREST,ASSY,STOP BRACKET,FLIP-UP ARMREST,SYNERGY/VERSA TILT,TEXTURED	1
I1	FRMASMB7045	MOUNT BRACKET,ASSY,CLAMP,FLIP-UP ARMREST,SYNERGY/VERSA TILT,TEXTURED	1
J1		BACK PLATE BRACKET,T-SLOT,1",C-1359-005	2
K1	SCRASBH1148	SCREW,BHSCS,BLACK ZINC,1/4-20 X 0.50",W/PATCH	2
L1	DWR1359B003	T-NUT,BLACK,1/4-20 X 0.563",0.50" TH,B-1359-003	2
M1	SCRSHFH1010	SCREW,SHFH,BLACK ZINC,1/4-20 X 0.75"	2
N1	PROECAP1029	END CAP,PLASTIC,1.25" DIA,FITS 11 GAUGE TUBE,RIBBED,C-1005-058	1
O1	WASWAVY1007	WASHER,BELLEVILLE,0.515" ID X 1.00" OD X 0.035" TH	1

2.1. Back Cane Mounted, Flip-up Arms

*** Individual components are not available for replacement.
The entire assembly must be purchased. ***

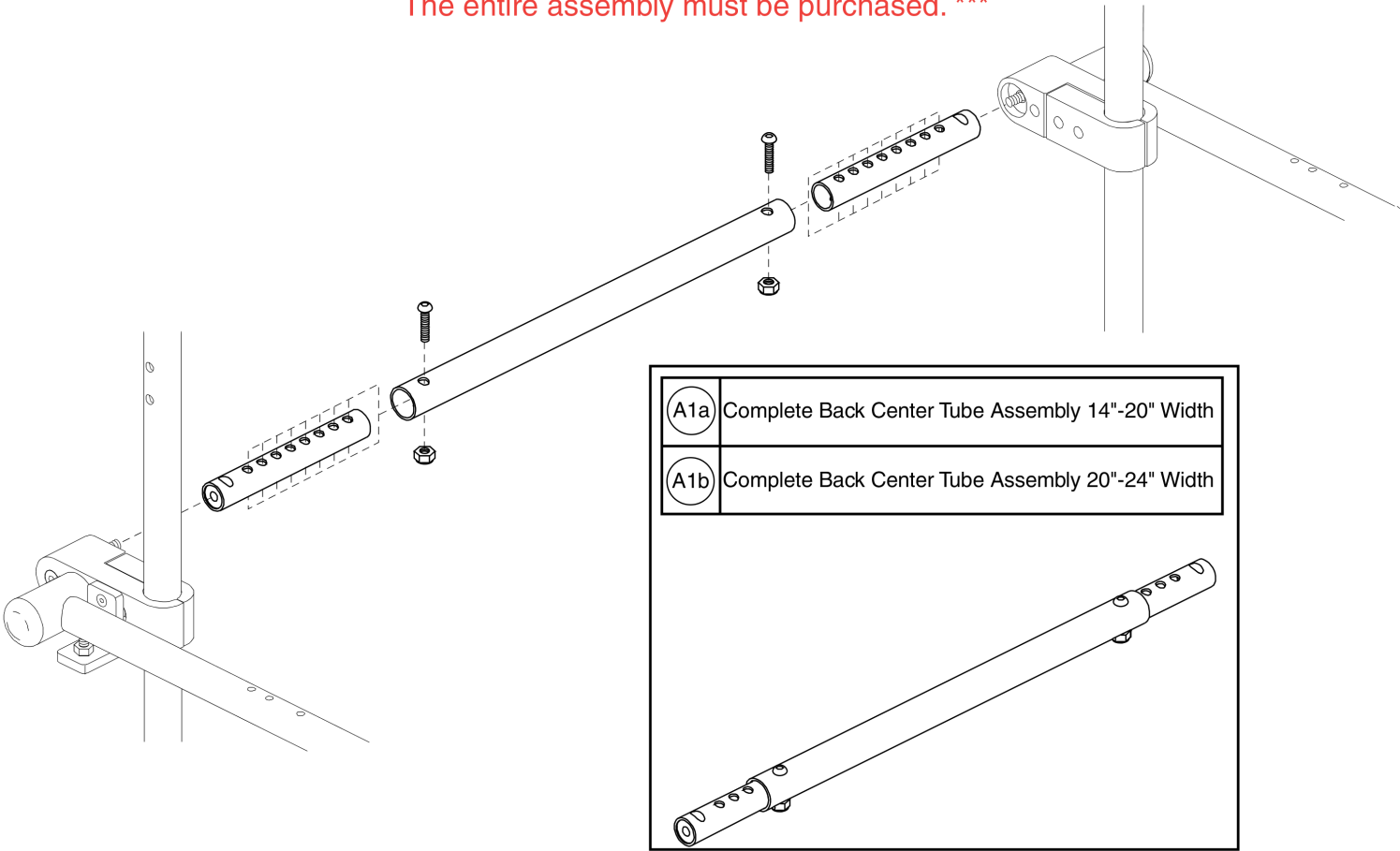

Back Center Tube - 13"W-24"W, Textured



Item	Part No.	Name	Qty
A1a	FRMASMB6600	BACK CENTER TUBE,ASSY,13"-20"W,SYNERGY,FLIP-UP ARMREST	1
A1b	FRMASMB6601	BACK CENTER TUBE,ASSY,20"-24"W,SYNERGY,FLIP-UP ARMREST	1

Appendix A:Pages Index

Back Center Tube - 13"W-24"W, Textured	5
Desk Length, Left / Right, Flip Up Armrest, Textured	1
Full Length, Left / Right, Flip Up Armrest, Textured	3

Appendix B:Parts Index

BLTASHL1017	2, 4
DWR1359B003	2, 4
DWR1359C005	2, 4
FRMASMB6600	6
FRMASMB6601	6
FRMASMB7038	4
FRMASMB7039	4
FRMASMB7040	2
FRMASMB7041	2
FRMASMB7042	2, 4
FRMASMB7045	2, 4
PROECAP1029	2, 2, 4, 4
SCRASBH1053	2, 4
SCRASBH1148	2, 4
SCRSHFH1010	2, 4
SETLRAM1115	2
SETLRAM1116	4
SETLRAM1164	2
SETLRAM1165	4
WASWAVY1007	2, 4

Illustrated Parts Breakdown

USA - Quantum Rehab

401 York Avenue
Duryea, Pa 18642
www.quantumrehab.com

USA - Pride Mobility Products Corp.

182 Susquehanna Avenue
Exeter, Pa 18643
www.pridemobility.com

Canada

5096 South Service Road
Beamsville, Ontario L0R 1B3
www.pridemobility.ca
www.quantumrehab.com

Australia

20-24 Apollo Drive
Hallam, Victoria 3803
www.pridemobility.com.au
www.quantumrehab.com.au

New Zealand

38 Lansford Crescent
Avondale, Auckland 0600
www.pridemobility.co.nz

UK

32 Wedgwood Road
Bicester, Oxfordshire OX26 4UL
www.pride-mobility.co.uk
www.quantumrehab.co.uk

Netherlands

(Authorised EU Representative)

De Zwaan 3
1601 MS Enkhuizen
www.pridemobility.eu
www.quantumrehab.eu

Italy

Via del Progresso, ang. Via del Lavoro
Loc. Prato della Corte
00065 Fiano Romano (RM)
www.pride-italia.it
www.quantumrehab-italia.it

France

26 Rue Monseigneur Ancel
69800 Saint-Priest
www.pridemobility.fr
www.quantumrehab.fr

Spain

Calle Las Minas Número 67
Polígono Industrial Urtinsa II, de Alcorcón
28923 Madrid

Germany

Hövelrieger Str. 28
33161 Hövelhof
www.pridemobility.de

China

Room 508, Building #4
TianNa Business Zone
No. 500 Jianyun Road
Pudong New Area
Shanghai 201318

