


TRUST
CARE




LET'S SHOWER

Article No. LSH101


x 1

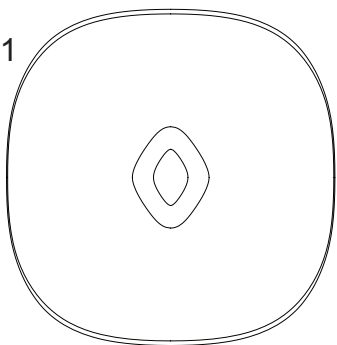

x 2

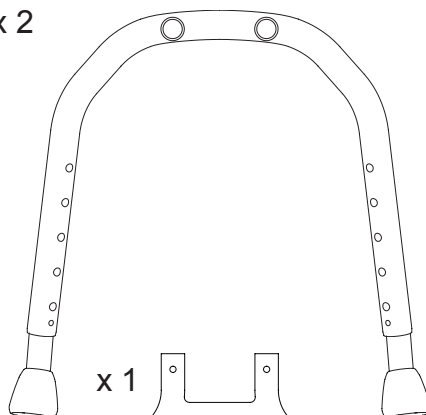


x 4

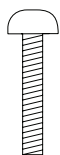


x 4






x 1

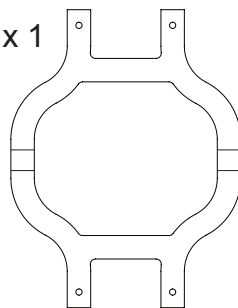

1

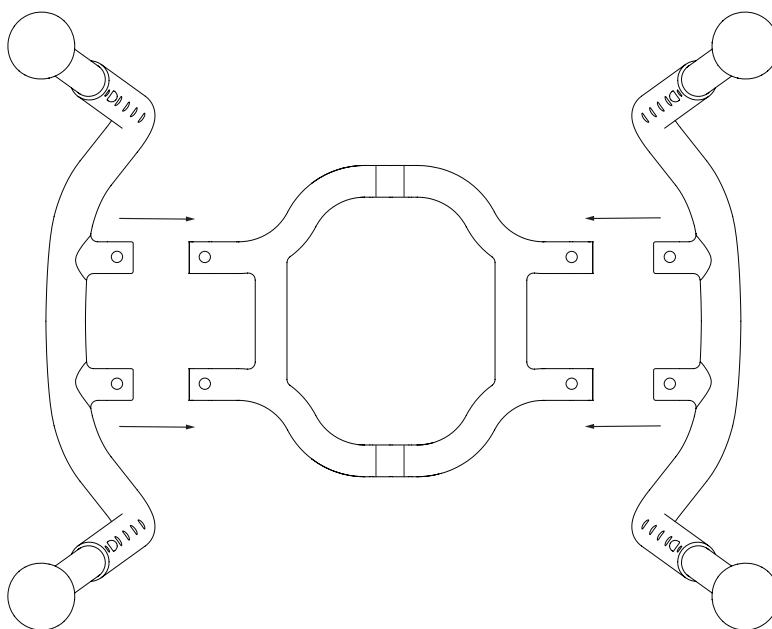

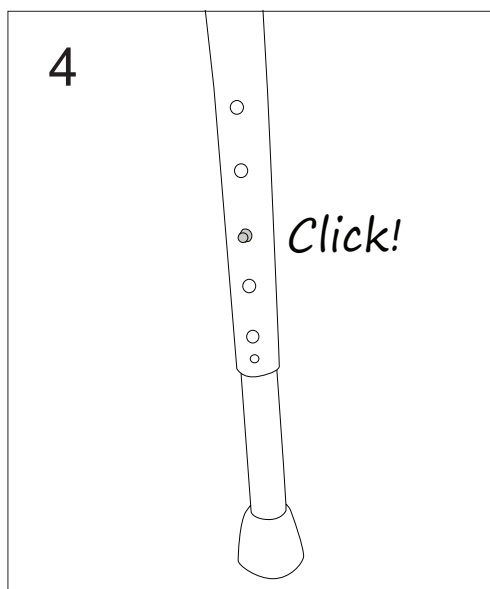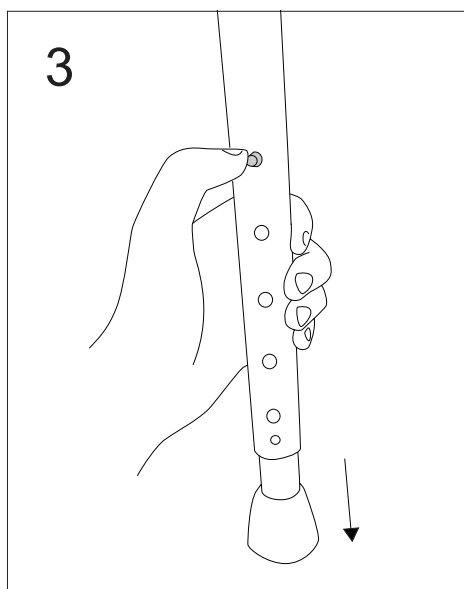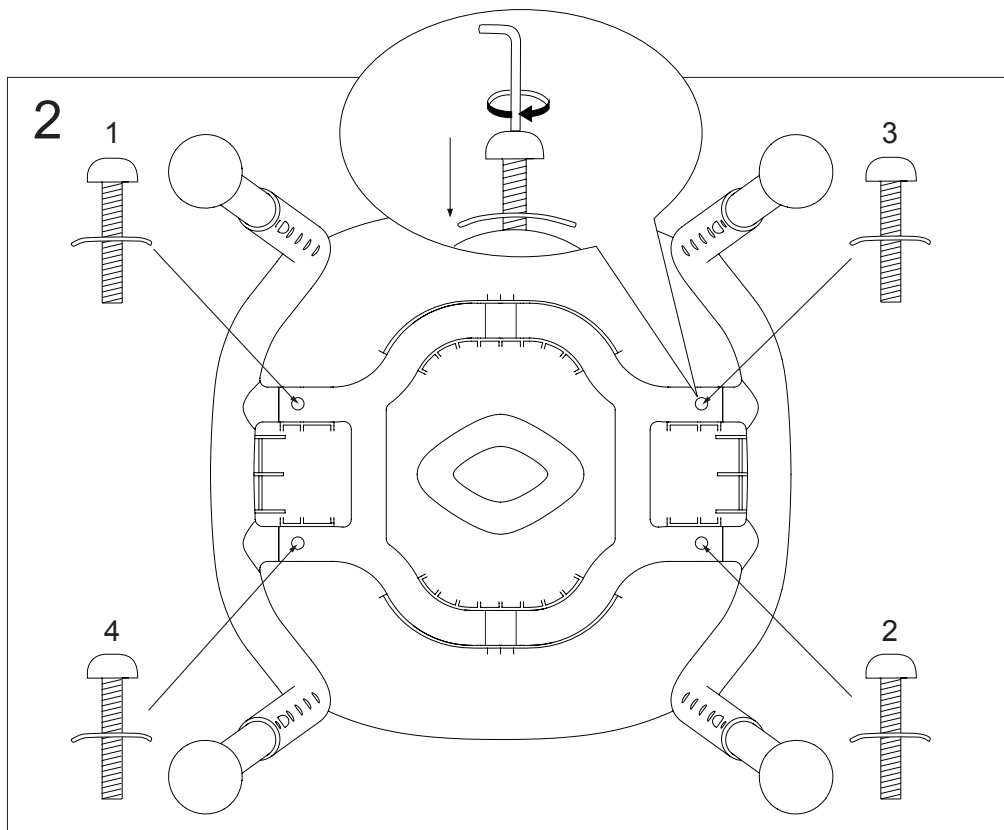

ENGLISH

Let's Shower is a new bathstool with a modern design to be used in the bathroom and shower. It has a beautiful and stable construction for safe and secure use. The height of Let's Shower can easily be adjusted to the desired position with the buttons on the legs.

Warning: Do not stand on the bathstool.

Warning: A defective product must not be used.

Warning: Always adjust all of the legs to the same height to avoid falling accidents because of the stool becoming unsteady.

Max user weight: Do not overload. Maximum user weight is 150 kg.

Cleaning: Use detergent without solvent, pH 5 – 9, or 70% disinfection spirit.

Decontamination: Use alkaline detergent pH min 6, in max 85°C/185°F for no more than 3 min.

Spare parts: Use only original accessories and original spare parts from Trust Care.



Technical data:

Material: Seat ABS, Tube frame aluminium, Feet TPE

Weight: 2,4 kg

Max user weight: 150 kg

Height: Adjustable from 45 to 53 cms.

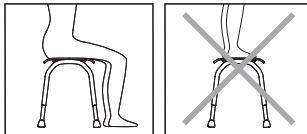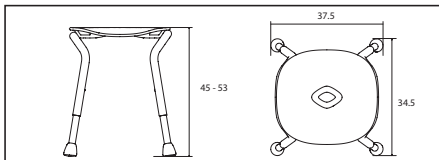

The stool is intended to sit on. Do not stand on the stool because of the risk of falling accidents.

