

TRUST
CARE



LET'S STEP

Article No. LST101

ENGLISH

A new footstool in modern, appealing design for use in bathrooms. The stool can be used as an aid to get in and out of the bathtub or as a support for your feet when you wash them.

Let's Step has a beautiful design for safe and secure use. It's easy to clean and only weighs 1.5 kg.

The ridges on the top of the footstool makes for a secure grip for your feet when the stool is wet.

Warning: A defective product must not be used. **Max user weight:** Do not overload. Maximum user weight is 150 kg.

Cleaning: Use detergent without solvent, pH 5 – 9, or 70% disinfection spirit.

Decontamination: Use alkaline detergent pH min 6, in max 85°C/185°F for not more than 3 min.

Spare parts: Use only original spare parts from Trust Care.

Technical data Let's Step:

Material: Seat ABS, Tube frame aluminium, Feet TPE

Weight: 1.6 kg

Max user weight: 150 kg

Height: 17 cm

Colour: Seat black, legs white, feet black

