

Quantum/Pride Outback Battery Access

Accessing Batteries on Quantum/Pride Outback

Before Starting: Turn off chair.

The procedure to access the batteries is the same with all seating types on both Quantum and Pride chairs.



Accessing Battery Procedure

Step 1. - Locate front lights mounted on either side of base.

Step2. - Remove screw located with arrow. Then gently pull the light toward you and off its mount. Let the light hang.

Step3. - Repeat step 2 on other side.



Step 4. - Find an 8mm T handle or long hex socket and insert through hole in body work into the hex bolt located behind.

Step 5. - Remove hex bolt and repeat on other side.



step 6. - Lift seating system by the foot platform. Lift all the way up until the collapsible support locks into place.

Step7. - Perform battery testing or replacement.

Step8. - While supporting the weight of the seating system. Push in spring clips on both sides of the collapsible support and push middle of support towards rear of chair, then slowly lower seating system back onto the base.

Step9. - Reverse all steps previous to step 6.



THANK YOU!