

## Instructions of Adjusting the Width of Lateral Support Bar


KARMA has 3 sizes of lateral support bars. Please choose suitable one to install your wheelchair by referring to the 2.3.1 matrix.

### 1、 Tools:


- 1.1. Hammer
- 1.2. M5 Screw

### 2. Steps:



Adjust the lateral support bar according to the distance, L1, between the handles.

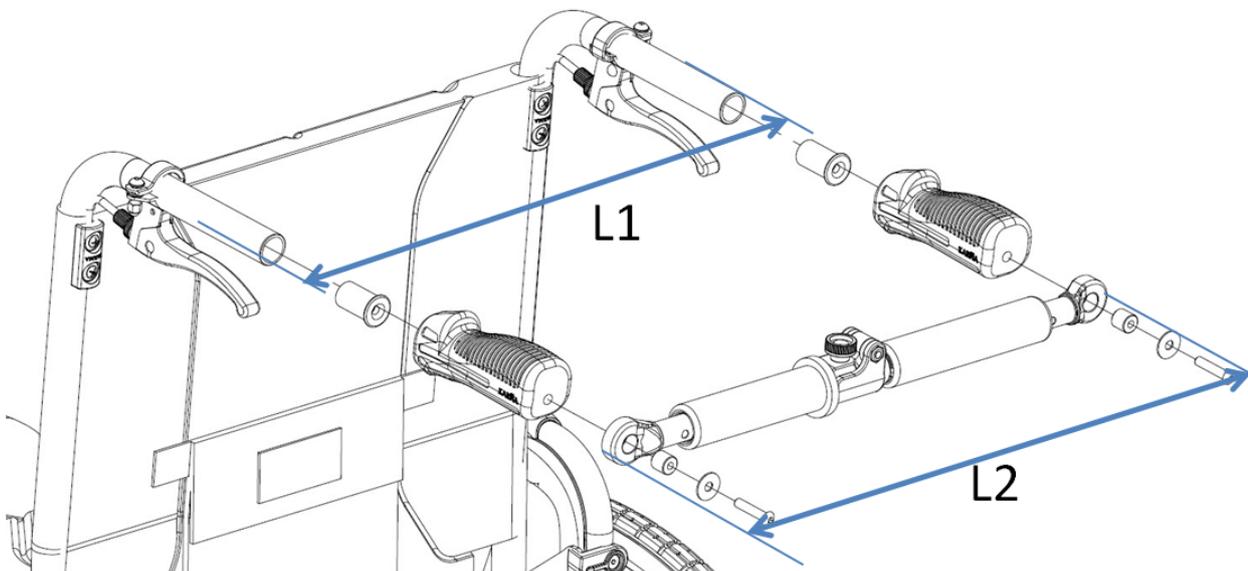




2.1. Push both lateral support bar Foam inwards until all four spring pins can be seen.

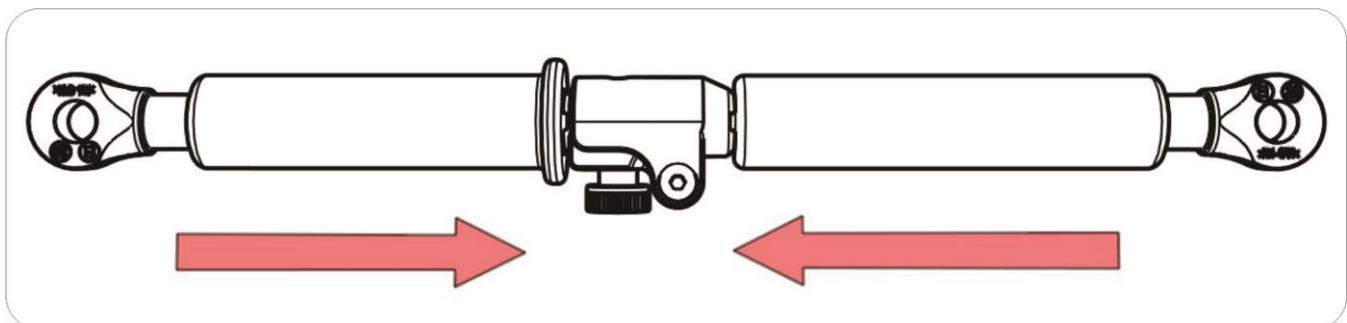

2.2. There are totally four spring pins, two on each side. Use a hammer and a M5 screw to take them out.

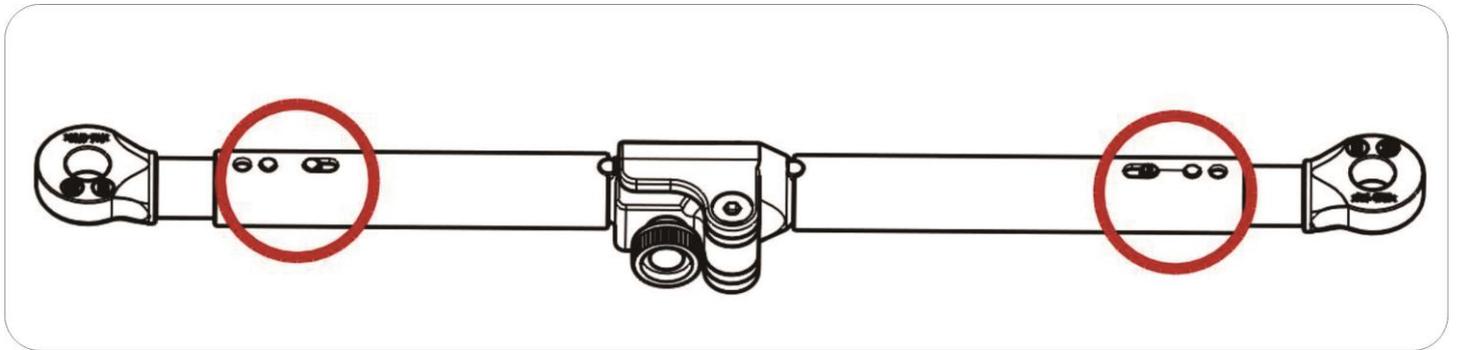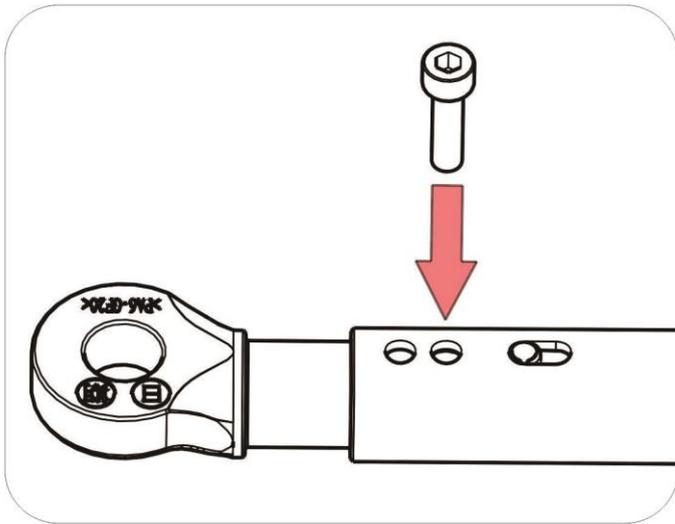

2.3. Select the locations of spring pins according to the desired seat width.

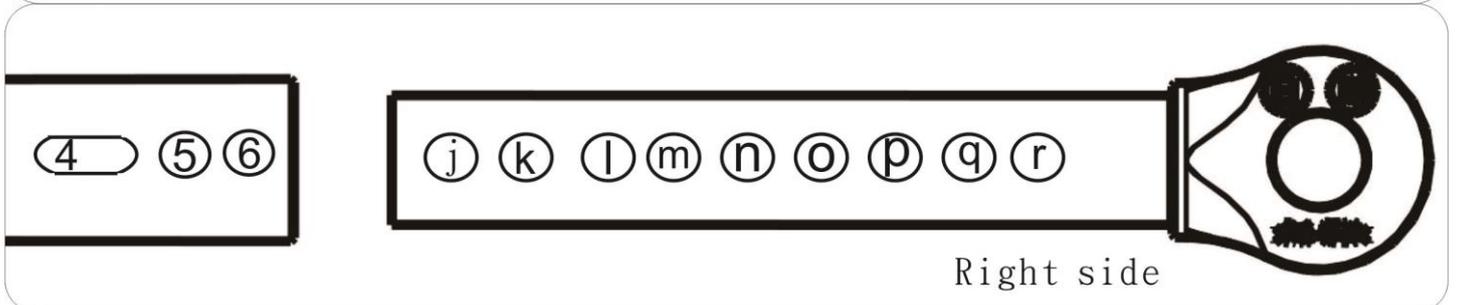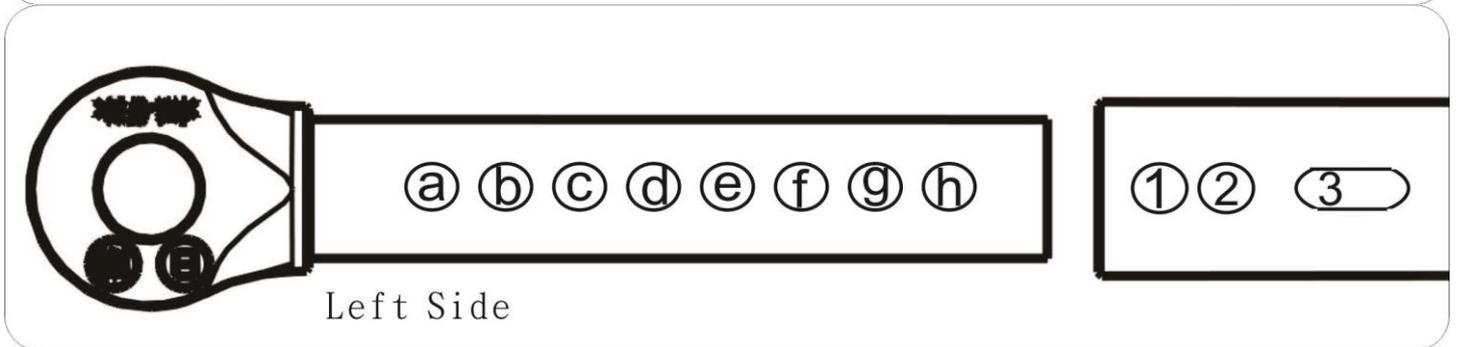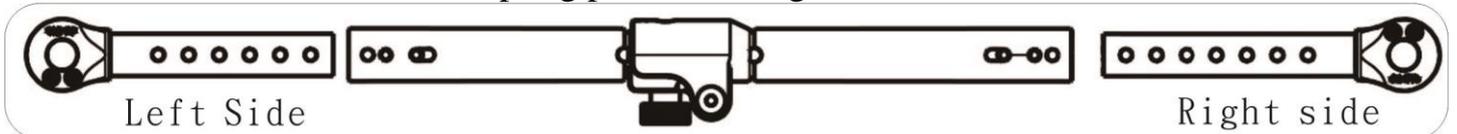

## 2.3.1


Part Number	The distance between the handles : L1	Adjustable length	The Length of Lateral Support Bar : L1		Locations of Spring Pins	
151200000101	343 mm	395~433.3	395 mm	Left Side	a	1
					c	3
				Right Side	r	4
					p	6
151200000102	368 mm	395~433.3	405.6 mm	Left Side	B	2
					C	3
				Right Side	q	5
					P	4
151200000102	394 mm	405.6~433.3	433.3 mm	Left Side	B	1
					D	3
				Right Side	P	6
					n	4
151200000103	420 mm	461.8~538	461.8mm	Left Side	b	2
					c	3
				Right Side	q	5
					P	4
151200000103	445 mm	461.8~538	489.5mm	Left Side	b	1
					d	3
				Right Side	p	6
					n	4
151200000103	470 mm	461.8~538	512.6 mm	Left Side	d	2
					e	3
				Right Side	o	5
					n	4
151200000103	495 mm	461.8~538	538 mm	Left Side	e	2
					f	3
				Right	n	5

				Side	m	4
--	--	--	--	------	---	---


- 2.4. Reinstall spring pins back to the lateral support bar by using a hammer and a M5 screw.  
Pull both lateral support bar Foam outwards until all four spring pins cannot be seen.

### 3 · Key Inspections:

- 3.1. Ensure all spring pins are appropriately installed.  
3.2. If the backrest tube is fordable, the knob nut shall be located on the top side.

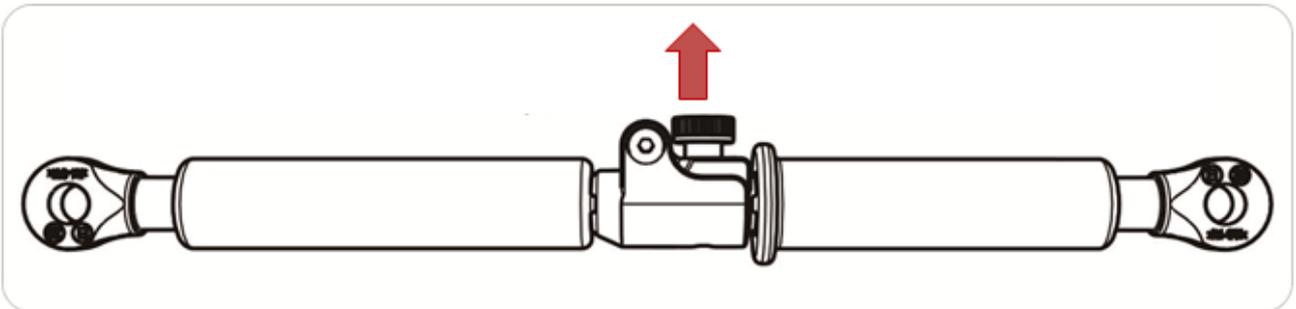

- 3.2. If the backrest tube is not fordable, the knob nut shall be located on the bottom side.

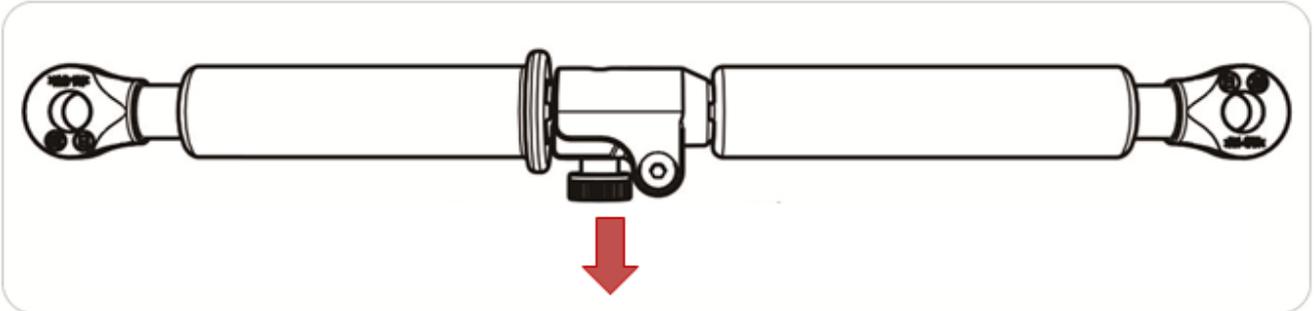

# **karma**

National agent:

Sold and serviced by:

**KARMA MEDICAL PRODUCTS CO., LTD.**

[Http://www.KarmaMedical.com](http://www.KarmaMedical.com)

E-mail:[globalsales@karma.com.tw](mailto:globalsales@karma.com.tw)

KARMA reserves the right to modify information herein without further notice.